



Batoonka Lagu bedelo aalada cabira heerka sonkorta dhiigga

Calaamada sheegta Heerka laga rabo dhiiga

Batoonka Lagu bedelo aalada cabira heerka sonkorta dhiigga


Batoonka laga saaro aalada muunad-akhriyaha

Godka la geliyo muunad-akhriyaha

Aalada muunad-akhriyaha



## Baarista sonkorta dhiiga

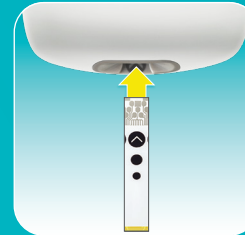


Markasta gacmahaaga ku dhaq biyo kulul, oo leh saabuun kadibna qalaji kahor inta aadan bilaabin baaritaanka. Aalada cabira heerka sonkorta dhiigga way isdaaraysaa markaad muunad-akhriyaha geliso aalada lagu cabiro heerka sonkorta dhiigga. Afka hore ee birta ah ee muunad-akhriyaha waxaa la geliyaa aalada lagu cabiro sonkorta.



Marka ay baxdo calaamada biligleynaya ee hoos ujanjeerta, fartaada ku mud aalada cabira heerka sonkorta dhiigga.

Qaybta dambe ka fiiri tilmaamaha sida loo isticmaalo aalada cabira heerka sonkorta dhiigga.

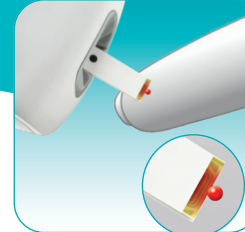

Taabo **cirifka jaalaha ah** ee muunad-akhriyaha ee dhobicda dhiig ah. Fartaada ka qaad muunad-akhriyaha marka caalamada biligleynaya ee saacada sheegaysa ay kasoo muuqato shaashada.



Natiijada baaritaanka waxay kasoo muuqanaysaa shaashada wax kayar 4 ilbiriqsi. Falaartu waxay sheegaysaa in natiijada baaritaankaaga ay sareeyo, joogto meeshii loogu talagalay, ama inay ka hooseeyo heerka laga rabay. Heerka laga rabo waxaa lagu jaangooyaa hadba yoolalka daaweyntaada gaarka ah.



Afka hore ee batoonka aalada akhriyahaheerka sonkorta dhiigga

Afka hore ee aalada cirbada waxaa kujira 6 cirbadood

Afka hore ee cirbada lagu mudo farta  
Fur la wareejin karo oo leh 11 wareeg



ACCU-CHEK, ACCU-CHEK INSTANT iyo FASTCLIX waa calaamadaha ganacsiga ee Roche Diabetes Care. © 2021 Roche Diabetes Care.

www.accu-chek.com | Roche Diabetes Care GmbH  
| Sandhofer Straße 116 | D-68305 Mannheim

## Marka aad isticmaalaysa aalada farta lagu muddo ee FastClix




ku dhaq gacmahaaga biyo kulul oo saabuun leh kahor inta aadan qaadin dhiiga.


Guud ahaan gacmahaaga qalaji.



Furka ka bixi aalada farta lagu mudo.

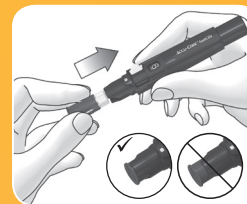

Ku xir afka aalada ee leh cirbada oo cusub - qaybta cad marka hore geli - aalada farta lagu mudo.

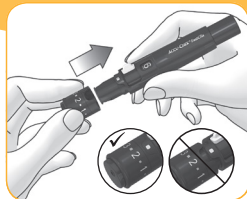

Afka hore ee cirbada si sax ah ayaa loo geliyay marka ayadoo dhan ay dhexgasho aalada cabirka, kaliya cirifka hore ayaa afka hore laga arki karaa, oo kaliya cirifka soo taagan ayaa ka soo baxa aaladda farta lagu mudo qaybta hore.

Bedel furka.



Wareeji furka illaa laga gaaro gunta diilinta kore oo leh calaamad muujiyaha.



Cirbada aalada cabirka ku mud meesha aad dooratay. Hoos u riix batoonka cirbada, cirbada ayaa soo baxaysa waxayna mudaa maqaarka.

Baar sonkorta dhiigaaga isla markiiba marka soo qaado dhiiga.