# Start Here

## Quick Start Guide

### WARNING

- This Quick Start Guide does not replace the User's Manual for your Accu-Chek Guide blood glucose meter. The User's Manual contains important handling instructions and additional information.
- The lancing device is intended for personal use only.

### Overview


- 1 Display
- 2 Back Button
- 3 Up Arrow and Down Arrow Buttons
- 4 Power/Set/OK Button
- 5 Test Strip Slot with Light
- 6 Battery Door
- 7 Micro USB Port
- 8 Test Strip Ejector

### Set Language and Time Format


Turn the meter on by briefly pressing **OK**. **Language** appears. Press **Up** or **Down** to highlight the desired language. Press **OK**.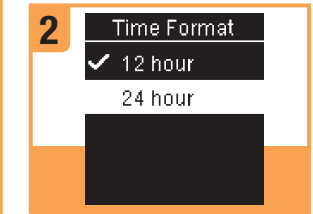


**Time Format** appears. Press **Up** or **Down** to highlight **12 hour** (am/pm) or **24 hour**. Press **OK** to set the option and return to **Main Menu**.


### Set up the Lancing Device

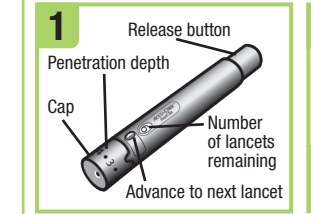
Familiarise yourself with the Accu-Chek FastClix lancing device.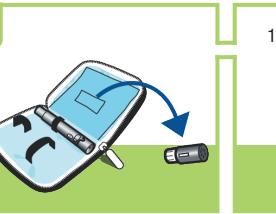


Get the lancet drum from the pocket in the carry case.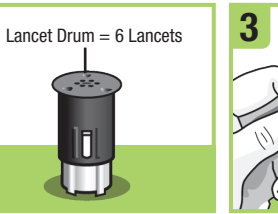


**NOTE**  
Each lancet drum contains 6 sterile lancets.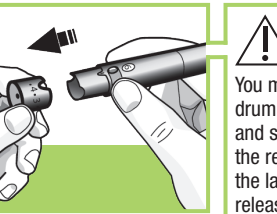


Remove the cap by pulling it straight off. Do not twist the cap.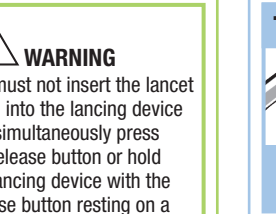


**WARNING**  
You must not insert the lancet drum into the lancing device and simultaneously press the release button or hold the lancing device with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.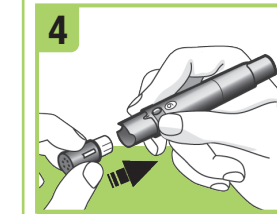


Insert the lancet drum, white end first, into the lancing device. Push the drum all the way in until it clicks firmly into place.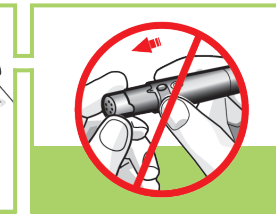

**NOTE**  
Do not remove the lancet drum until all 6 lancets are used since the drum cannot be reused once it has been removed from the lancing device.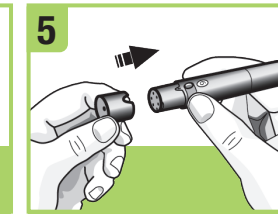

Replace the cap on the lancing device. Make sure the notch on the cap lines up with the notch on the lancing device.

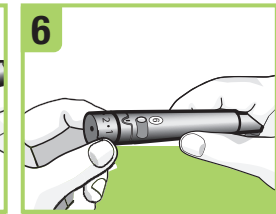
Adjust the lancet depth to a comfortable penetration level. Start at 2. For tougher skin, dial to a higher number.

Using the lancing device: By pressing the release button, the lancing device primes and pricks with 1 click.

### Perform a Blood Glucose Test

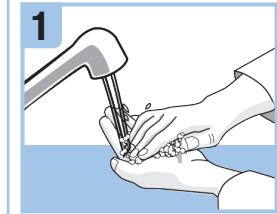
Wash and dry your hands before you perform a blood glucose test.


Check the use by date on the test strip container. Do not use test strips past the use by date.


Remove a test strip from the test strip container. Close the cap tightly.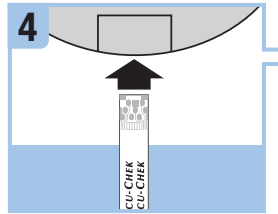


Insert the metallic end of the test strip into the meter.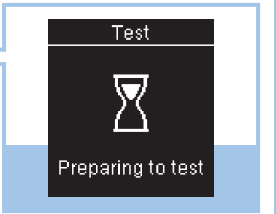


The meter turns on. **Preparing to test** appears.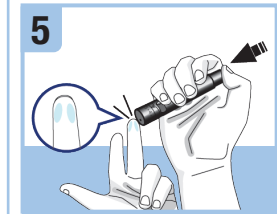


Press the lancing device firmly against the side of your fingertip and press the release button to prick your finger.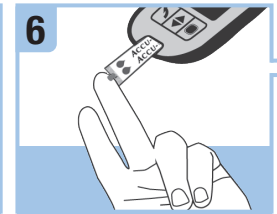

Touch the **yellow edge** of the test strip to the blood drop. Remove your finger from the test strip when **Analyzing** appears. Do not put blood on top of the test strip.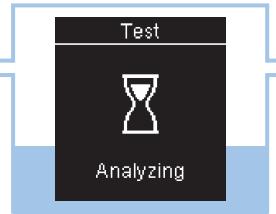

The test result appears on the display in less than 4 seconds.

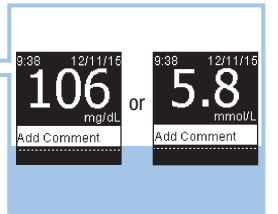The test result appears on the display in less than 4 seconds.

Prepare the lancing device for the next test. Slide the lever over and back once to advance to the next lancet. The window shows the number of lancets remaining.

### Download the free Accu-Chek Connect diabetes management app\* at:

\*Available for select mobile devices.

LAST UPDATE: 2015-09

Roche Diabetes Care GmbH  
Sandhofer Strasse 116  
68305 Mannheim, Germany  
www.accu-chek.com

ACCU-CHEK, ACCU-CHEK CONNECT, ACCU-CHEK GUIDE, and FASTCLIX are trademarks of Roche.

APPLE, APPLE logo, and APP STORE are trademarks of Apple Inc.

GOOGLE PLAY is a trademark of Google Inc.

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© 2016 Roche Diabetes Care